

| HUSH Curtain®


MATERIALS & SUPPLIES

1. Additional roller carriers (new roller carriers are recommended for ease of movement)
2. Flathead screwdriver to add extra roller carriers to track
3. Silicon grease or spray to lubricate track
4. Ladder


REPLACE EXISTING CURTAINS

HUSH Curtain® easily installs by hanging on the already existing hospital curtain track. It is the responsibility of the facility to install the cubicle track per the manufacturer's direction regarding ceiling attachment and support. Simply remove the hospital curtain and hang HUSH Curtain in its place. Once all panels are hung zip them together to create one seamless curtain.


1. Start hanging panels with the roller carrier closest to the wall.
2. One end of each panel has a zipper pull, while the other end has only zipper teeth.

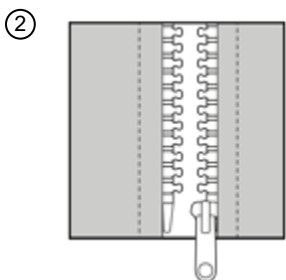

3. Place the end with only teeth against the wall when starting install. That will ensure the zipper pull is available to attach the next panel.

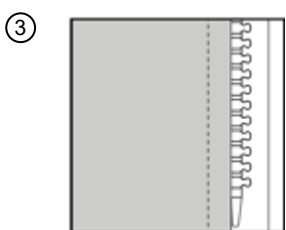

4. When hanging each curtain, make sure that you coordinate the teeth and the zipper pull with the previous curtain, so when install is done, they can zip together.

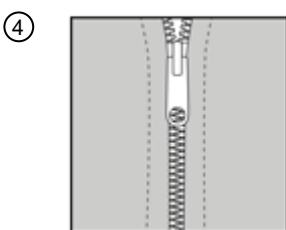

HUSH Curtain®

5. Overlap the end grommet of each panel onto one roller carrier for ease of movement.

6. Position one free-floating roller carrier between the overlapping grommets so one is available if the panels ever need to separate.

